

EasiCath® catheter and Set

Single-use hydrophilic coated catheter for intermittent catheterization

- Hydrophilic coated catheter for smooth passage
- Rounded, soft catheter eyes for extra comfort
- Set – integrated system with water ampoule, all-in-one
For intermittent catheterization anytime and anywhere



EasiCath[®] catheter and Set

EasiCath comes in a sterile packaging and is ready for use in less than a minute. It is completely atoxic and easy to use

Minimises friction

Filling the catheter bag with water and waiting 30 seconds bring out the benefits of the hydrophillic coating. The PVP binds the water molecules to the surface of the catheter, producing a smooth, slippery film that allows it to pass through the urethra with minimal discomfort and friction.

The difference in friction between the EasiCath catheter and an uncoated plastic catheter lubricated with gel can be felt. In addition to offering greater comfort, EasiCath reduces the likelihood of friction and urethral trauma.

Simple and easy handling

The EasiCath catheter is packaged in a sealed but easy-to-open bag. The top of the bag is supplied with a die-cut ring as well as an adhesive dot (peel off the protective cover), both of which can be used to suspend the bag when it is filled with water.

Gentle on urethral mucosa

The catheter eyelets of the EasiCath are soft, rounded and hydrophillic coated. This makes the catheter gentle on the urethral mucosa and the catheterization procedure more comfortable and safe.



EasiCath with LuerLock

The EasiCath is also available with a LuerLock instead of the standard connector. This allows the use of a syringe to ensure safe intravesical medication. The syringe can be connected before or after the catheter is inserted into the bladder.

EasiCath Set

EasiCath Set is an integrated system that includes everything needed for gentle intermittent catheterization: catheter, collection bag and sterile saline water solution. With this three-in-one product the user has everything needed to perform a catheterization any place and at any time.

Collection bag

The design and flexibility of the collection bag make it easy to return the catheter into the bag. The bag is provided with a volume indicator, a handle and an emptying facility.



Sterile saline solution

The ampoule contains a 0,9% sodium chloride solution (30 ml for men and 15 ml for women and children). Thus optimum water quality is always at hand, no matter where you are. The ampoule is made of polyethylene.

A complete system

The range also includes Conveen urisheaths and urine bags. Together with the EasiCath catheter they offer a complete system of high-quality appliances for people with bladder emptying problems.

Product compositions:

EasiCath: Polyvinyl chloride (PVC), DEHP, Polyvinylpyrrolidone (PVP).

EasiCath Luerlock: Polyvinyl chloride (PVC), DEHP, Polyvinylpyrrolidone (PVP), Polyethylene (PE).

EasiCath Set: Polyvinyl chloride (PVC), DEHP, Polyvinylpyrrolidone (PVP), Polyethylene (PE), Polypropylene (PP).

Product range

EasiCath for men and women



Size CH	Men Nelaton	Men Tieman	Female
10	05350	05380	05370
12	05352	05382	05372
14	05354	05384	05374
16	05356	05386	05376
18	05358	05388	
22	05362		

Units per box: 25

EasiCath with Luerlock



Size CH	Men Luerlock	Female Luerlock
08	05288	05268
10	05290	05270
12	05292	05272
14	05294	05274
16	05296	

EasiCath for children

Size CH	Paediatric
08	05008
10	05010
12	

Units per box: 25

EasiCath Set



Size CH	Male	Female	Paediatric
08	28005		28012
10	28006	28002	28013
12	28007	28003	
14	28008	28004	
16	28009		
18	28010		

Units per box: 20

Ostomy Care
Urology & Continence Care
Wound & Skin Care



Coloplast develops products and services that make life easier for people with very personal and private medical conditions. Working closely with the people who use our products, we create solutions that are sensitive to their special needs. We call this intimate healthcare. Our business includes ostomy care, urology and continence care and wound and skin care. We operate globally and employ more than 7,000 people.

The Coloplast logo is a registered trademark of Coloplast A/S. © [2011-08.]
All rights reserved Coloplast A/S, 3050 Humlebæk, Denmark.

Coloplast A/S
Holtedam 1
3050 Humlebæk
Denmark

www.coloplast.com